# CE50™ Heart Mobile

## User Guide

Clarity, a Division of Plantronics, Inc.  
4289 Bonny Oaks Drive, Chattanooga, TN 37406  
Tel: 800-426-3738 Fax: 800-325-8871  
E-mail: [claritycs@plantronics.com](mailto:claritycs@plantronics.com) Website: [www.clarityproducts.com](http://www.clarityproducts.com)

©2010 Clarity, a Division of Plantronics, Inc. All rights reserved. Clarity and CE50 are trademarks or registered trademarks of Plantronics, Inc. The Bluetooth trademark is owned by Bluetooth SIG, Inc. and any use of the mark by Plantronics, Inc. is under license.  
Rev. C (06-10)

Table of contents

Package content	1
Important safety and operational information	2
Charging warnings	4
Battery warnings	5
Functions	6
Setup	7
Wireless setup for mobile telephone	7
Installing neckloop	7
Operation	8
Indicators	8
Charging	8
Adjusting volume	8
Accepting an incoming call	9
Making a mobile telephone call	9
Rejecting an incoming call	9
Cleaning and maintenance	9
Regulatory notices	10
Warranty	12

**Package content**

CE50 HearIt Mobile

Power supply

Congratulations and thank you for purchasing CE50 HearIt Mobile.  
 CE50 HearIt Mobile is a hands-free, neckloop solution for amplifying sound from your Bluetooth mobile phone.  
 CE50 HearIt Mobile has been specially designed for people with impaired hearing who want to use a mobile phone hands-free – and without any compromise in sound quality.

## General

Please read the following safety and operational instructions before using this product. Please keep these instructions for your reference. When using this product, these basic safety precautions and warnings should be followed to reduce the risk of fire, electric shock, injury to persons and damage to property. Additional warnings may be present on the product, packaging or in the user instructions.

Operating and storage temperature is 0°C to +45°C (32°F to +113°F).

Exposure to high volume sound levels or excessive sound pressure may damage your hearing. Although there is no single volume setting that is appropriate for everyone, you should always use your CE50 HearIt amplifier with the volume set at moderate levels and avoid prolonged exposure to high volume sound levels. The louder the volume, the less time is required before your hearing could be affected. You may experience different sound levels when using your CE50 HearIt amplifier with different devices. The device you use and its settings affect the level of sound you hear. If you experience hearing discomfort, you should stop listening to the device through your CE50 HearIt amplifier. To protect your hearing, some hearing experts suggest that you:

1. Set the volume control in a low position before using the CE50 HearIt amplifier.
2. Limit the amount of time you use your CE50 HearIt amplifier at high volume.
3. Avoid turning up the volume to block out noisy surroundings.
4. Turn the volume down if the sound from the CE50 HearIt amplifier prevents you from hearing people speaking near you.

Misuse of this product can result in sudden and permanent hearing loss.

Do not disassemble the product or charger as this may expose you to dangerous voltages or other risks. Incorrect reassembly can cause electric shock or fire when the product or charger is subsequently used.

Do not insert anything into the product or charger because this may cause damage to the internal components or cause injury to you.

Avoid contact with liquids. Do not locate this product near water, for example, near a bathtub or sink, in a wet basement, or near a swimming pool to protect against damage to the product or injury to you.

Discontinue use of product and contact Clarity if the product overheats, has a damaged cord or plug, if the product has been dropped or damaged, or if the product has come into contact with liquids.

If you experience a skin irritation after using this product, discontinue use and contact Clarity.

CE50 HearIt Mobile is not a toy and therefore should be kept out of the reach of children or others with learning difficulties who may swallow parts or otherwise cause injury to themselves. If a part is swallowed, see a doctor immediately.

Never allow others to wear or use your CE50 HearIt Mobile as incorrect/wrongful usage could cause permanent damage to their hearing.

### *Interference*

Your CE50 HearIt Mobile amplifier has been thoroughly tested for interference according to the most stringent international standards. However, new technical developments constantly bring new products into society, and some may emit electromagnetic radiation, leading to unforeseen interference with the CE50 HearIt Mobile amplifier. Examples include induction cooking appliances, alarm systems, mobile telephones, fax machines, personal computer devices, X-rays, computed tomography, etc.

### *Use on Aircraft*

The CE50 HearIt Mobile should not be used on board aircraft unless specifically permitted by the flight personnel.

### *Implantable Devices*

CE50 HearIt Mobile is designed to comply with the most stringent Standards of International Electromagnetic Compatibility. However, CE50 HearIt Mobile may cause interference with other medical devices such as defibrillators and pacemakers. Please contact the manufacturer of your implantable device for information about the risk of disturbance. Such interference can also be caused by power line disturbances, airport metal detectors, electromagnetic fields from other medical devices, radio signals and electrostatic discharges.

Never expose CE50 HearIt Mobile to extreme heat, such as in an oven or in a fire. There is a risk that it will explode and cause serious injury.

The Specific Absorption Rate (SAR) level for the CE50 HearIt Mobile is well below international limits for human exposure. For comparison, the SAR level for the CE50 HearIt Mobile is equal to that of a Bluetooth headset for a mobile telephone.

### *Avoid Heat and Chemicals*

Your CE50 HearIt Mobile must never be exposed to extreme heat, e.g., left inside a parked car in the sun. The CE50 should never be dried in microwave ovens or other ovens. CE50 HearIt Mobile must not be exposed to or cleaned with cleaning chemicals or alcohol.

### *Use in Cars*

Using the CE50 HearIt Mobile with a mobile phone while driving may distract you. If driving conditions demand it and when safe to do so, pull off the road and park before making and answering calls. Check local laws regarding use of a handsfree device while driving. If you use the CE50 amplifier while driving, ensure your attention and focus remain on driving safely.

*Use of the Neckloop*

If by accident the neckloop should get caught in something while wearing it, the neckloop is designed to open automatically. Therefore, never modify the neckloop in any way. The neckloop should not be used for children who weigh less than 11 kg (approximately 24 lbs.).

Observe all signs and instructions regarding use of an electrical device or RF radio product in designated areas, such as gas stations, hospitals, blasting areas, potentially explosive atmospheres or aircraft.

Keep all products, cords and cables away from operating machinery. As with all corded products, there is a danger of strangulation if a cord should become wrapped around your neck.

## Charging Warnings

To reduce the risk of electric shock, explosion or fire, use only the charger supplied by Clarity to charge the product. Do not use the charger with other products or for any other purpose. Ensure that the voltage rating corresponds to the local mains or power supply you intend to use.

Charge the CE50 amplifier according to the instructions supplied with the unit. Ensure the mains (AC Power) socket-outlet for the charger is located near the equipment and is easily accessible.

## Battery Warnings

To avoid the risk of explosion, fire or leakage of toxic chemicals, please observe the following warnings:


Do not dispose of the product or battery in a fire. The battery cells may explode.

This product has a non-replaceable battery; do not attempt to open the product or remove the battery. The battery contains corrosive materials which can cause damage to eyes or skin and may be toxic if swallowed.

Your product and batteries must be recycled or disposed of properly. Contact your local recycling center for information on proper disposal.

The following switch mode power supply can be used with the CE50 HearIt Mobile. Manufacture: Golden Profit, MODEL No.: GPE060A-06100-1.

- R1: On/off switch, answering button, and Bluetooth® indicator
- R2: Volume button +/- (up/down)
- R3: Bluetooth and volume level 1 indicator
- R4: Volume level 2 indicator
- R5: Battery and volume level 3 indicator
- R6: Volume level 4 indicator
- R7: Volume level 5 indicator
- R8: Headphones jack
- R9: Neckloop jack
- R10: Neckloop
- R11: Charging jack
- R12: Microphone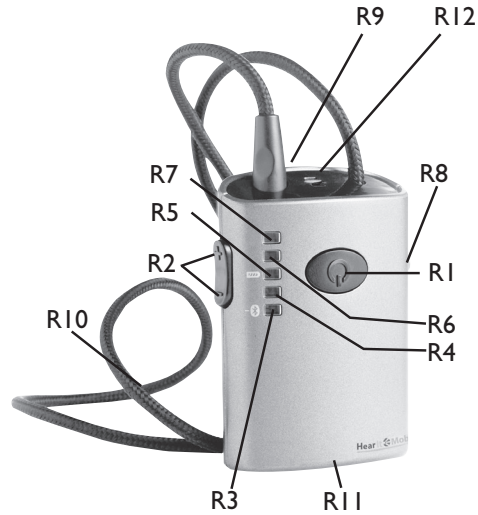


Button/diode/jack	Functions
R1	<ul style="list-style-type: none"> <li>• Power on (hold for up to five seconds)</li> <li>• Power off (hold for up to five seconds)</li> <li>• Answering and ending mobile telephone calls</li> <li>• Blue light: Bluetooth active</li> <li>• Blue blink: Incoming call</li> </ul>
R2+ (up)	<ul style="list-style-type: none"> <li>• Volume up</li> </ul>
R2- (down)	<ul style="list-style-type: none"> <li>• Volume down</li> <li>• Reject mobile telephone call (hold for two seconds)</li> </ul>
R3	<ul style="list-style-type: none"> <li>• Blue blink: Bluetooth not connected</li> <li>• Green light: Volume level 1</li> </ul>
R4	<ul style="list-style-type: none"> <li>• Green light: Volume level 2</li> </ul>
R5	<p>When in use</p> <ul style="list-style-type: none"> <li>• Red blink: Low battery</li> <li>• Green light: Volume level 3</li> </ul> <p>When charging</p> <ul style="list-style-type: none"> <li>• Red light: Charging</li> <li>• Green light: Fully charged</li> </ul>
R6	<ul style="list-style-type: none"> <li>• Green light: Volume level 4</li> </ul>
R7	<ul style="list-style-type: none"> <li>• Green light: Volume level 5</li> </ul>
R8	<ul style="list-style-type: none"> <li>• Jack for headphones</li> </ul>
R9	<ul style="list-style-type: none"> <li>• Jack for neckloop</li> </ul>
R10	<ul style="list-style-type: none"> <li>• Neckloop</li> </ul>
R11	<ul style="list-style-type: none"> <li>• Jack for charger</li> </ul>
R12	<ul style="list-style-type: none"> <li>• Microphone</li> </ul>

## Wireless setup for mobile telephone

Establishing a wireless connection between the Bluetooth mobile telephone and the CE50 HearIt Mobile

1. Turn CE50 HearIt Mobile on by holding down the receiver button **R1** for up to five seconds.
2. Enter the mobile telephone menu to allow the mobile telephone to search for available Bluetooth devices.
3. Select the Bluetooth entry "HearIt Mobile".
4. Enter the pass code into the mobile telephone (**0000**). Select "connect." A short beep will be heard in the hearing aid or in the headphones.
5. Once the wireless connection is established, receiver indicator **R3** stops blinking blue.

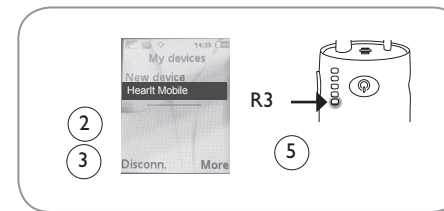Please note:

- Once a wireless connection between CE50 HearIt Mobile and a Bluetooth unit has been established, the units will automatically connect when they are within range of each other.
- During telephone conversations, the receiver indicator R1 will illuminate blue indicating that the wireless connection is active.

## Installing neckloop

In order to use CE50 HearIt Mobile with hearing aids in T-position (telecoil), the hearing aid must be equipped with a telecoil fitted by a hearing care professional.

Installing neckloop

1. Detach neckloop R10 from receiver jack R9 by pulling the loop wire.
2. Place the CE50 HearIt Mobile on the chest and place loop wire around the neck.
3. Reattach neckloop R10 to receiver jack R9.
4. Switch hearing aid to T (telecoil only) or MT (microphone and telecoil simultaneously) position.

**NOTE:** Neckloop and T/MT function on hearing aids.

Please note:

- Connecting headphones to the headphones jack R8 disconnects the signal to the neckloop.

## Indicators


Below you can see an overview of the light indicators on CE50 HearIt Mobile

Flashing blue (R1):	Incoming call
Blue (R1):	Wireless Bluetooth call activated
Flashing blue (R3):	No Bluetooth connection
Green (R3-R7):	Volume level
Flashing red (R5):	Battery low

## Charging

If the CE50 HearIt Mobile light indicator R5 flashes red during operation, or if the unit cannot be turned on, CE50 HearIt Mobile requires recharging.

1. Connect the charger wire to CE50 HearIt Mobile's jack R1 I.
2. Connect the charger to a power source and turn on.
3. The light indicator R5 lights continuously red during recharging.
4. When CE50 HearIt Mobile is fully recharged, light indicator R5 lights continuously green.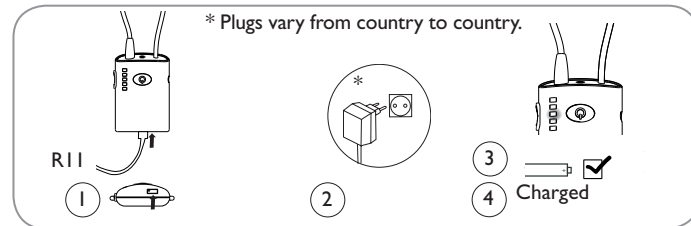


Please note:

- We recommend fully charging batteries before use. However, partly charging the units does not inflict damage to them.

## Adjusting volume

1. Press receiver button R2+ (volume up) or R2- (volume down) to set comfortable listening level.
2. The receiver indicators R3 to R7 illuminate depending on volume level.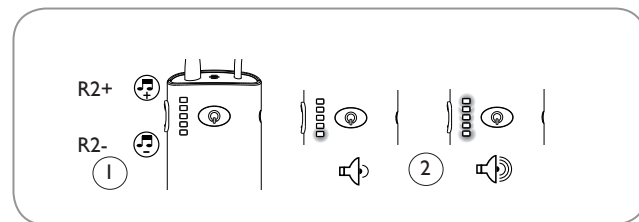


Please note:

- The maximum volume level has been reached if all five indicators R3 to R7 are illuminated. If the signal is still too weak, adjust the volume on your mobile telephone or contact your hearing care professional for adjustment of the telecoil in the hearing aid.

## Accepting an incoming call

1. When a call comes in, a ringing tone will be heard in the hearing aid or headphones and the receiver indicator **R1** starts blinking blue.
2. Press the button **R1** to accept the call.
3. Press the button **R1** to end the call.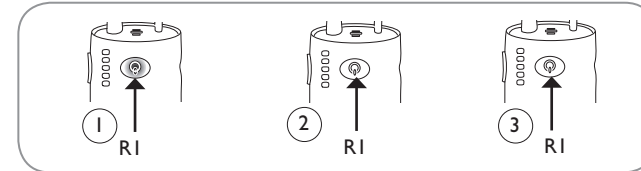

Please note:


- Only make a single short push to accept or end a call. It may take a couple of seconds before a call is completely connected or disconnected.
- The ringing tone of the mobile telephone call may be the same sound as heard from the mobile telephone itself depending on the mobile telephone used.
- If CE50 HearIt Mobile is turned off when a mobile telephone call comes in, that specific call cannot be answered or ended using CE50 HearIt Mobile.

## Making a mobile telephone call

1. Turn on CE50 HearIt Mobile by pressing the button R1 for up to five seconds.
2. Enter the dial number on the mobile telephone and pick up the handset.
3. The call is automatically transferred to CE50 HearIt Mobile.
4. To end the call, press the button R1 or end the call on the mobile telephone.

## Rejecting an incoming call

1. When a call comes in, a ringing tone will be heard in the hearing aid or headphones and the receiver indicator R1 starts blinking blue.
2. Press receiver button R2- (volume down) for two seconds to reject the call.

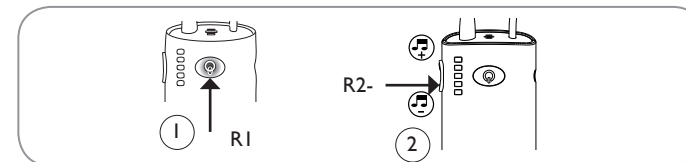## Cleaning and maintenance

1. Any dirt, moisture or grease should be cleaned by using a dry or damp cloth.

## Part 15 of FCC Rules Information

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Your equipment has been tested and found to comply with the limits of a Class B digital device, pursuant to Part 15 of FCC rules. These limits are designed to provide reasonable protection against harmful interference in residential installation.

This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation; if this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, you are encouraged to try to correct the interference by one of the following measures:

1. Where it can be done safely, reorient the receiving television or radio antenna.
2. To the extent possible, relocate the television, radio or other receiver with respect to the telephone equipment. (This increases the separation between the telephone equipment and the receiver.)
3. Connect the telephone equipment into an outlet on a circuit different from that to which the television, radio, or other receiver is connected.
4. Consult the dealer or an experienced Radio/TV Technician for help.

**WARNING:** Changes or modifications not expressly approved by the manufacturer responsible for compliance could void the user's authority to operate the equipment.

## Exposure to radio frequency radiation

This device and its antenna must not be collocated or operated in conjunction with any other antenna or transmitter. To comply with FCC RF exposure requirements, only use supplied antenna. Any unauthorized modification to the antenna or device could void the user's authority to operate this device.

## This equipment complies with Industry Canada.

**NOTICE:** The Canadian Department of Communications label identifies certified equipment. This certification means that the equipment meets certain telecommunications network protective operational and safety requirements. The Department does not guarantee that the equipment will operate to the user's satisfaction.

Before installing this equipment, users should ensure that it is permissible to be connected to the facilities of the local telecommunications company. The equipment must also be installed using an acceptable method of connection. The customer should be aware that compliance with the above conditions may not prevent degradation of service in some situations.

Repairs to certified equipment should be made by an authorized Canadian maintenance facility designated by the supplier. Any repairs or alterations made by the user to this equipment, or equipment malfunctions, may give the telecommunications company cause to request the user disconnect the equipment. Users should ensure for their own protection that the electrical ground connections of the power utility, telephone lines and internal metallic water pipe system, if present, are connected together. This precaution may be particularly important in rural areas.

**Caution:** Users should not attempt to make such connections themselves, but should contact the appropriate electric inspection authority, or electrician, as appropriate.

## Warranty

A One-year limited warranty is issued to the original purchaser of this product from Clarity.

**Incidental or Consequential Damages:** Neither Clarity nor your retailer dealer or selling distributors has any responsibility for any incidental or consequential damages including without limitation, commercial loss or profit, or for any incidental expenses, expenses, loss of time, or inconvenience. Some states do not allow exclusion or limitation of incidental or consequential damage, so the above limitation or exclusion may not apply to you.

**Other Legal Rights:** This warranty gives you specific legal rights and you may also have other rights which vary from state to state.

To obtain warranty service, please prepay shipment and return the unit to the appropriate facility listed below.

Clarity Service Center  
4289 Bonny Oaks Drive  
Chattanooga, Tennessee 37406  
Tel: 423-629-3500 or 800-426-3738  
Fax: 423-622-7646 or 800-325-8871

Plantronics Service Centre  
2732 Etienne Lenoir  
Laval, Quebec H7R 0A3  
Tel: 800-540-8363 or 514-956-8363  
Fax: 514-956-1825

Please use the original container, or pack the unit(s) in a sturdy carton with sufficient packing material to prevent damage. Include the following information:

- A proof-of-purchase indicating model number and date of purchase;
- Bill-to address;
- Ship-to address;

- Number and description of units shipped;
- Name and telephone number of person to call, should contact be necessary;
- Reason for return and description of the problem.

Damage occurring during shipment is deemed the responsibility of the carrier, and claims should be made directly with the carrier.

The following warranty and service information applies only to products purchased and used in the U.S. and Canada. For warranty information in other countries, please contact your local retailer or distributor.

**Limited Warranty:** Clarity, a division of Plantronics, Inc. ("Clarity") warrants to the original consumer purchaser that, except for limitations and exclusions set forth below, this product shall be free from defects in materials and workmanship for a period of one (1) year from the date of original purchase ("Warranty Period"). The obligation of Clarity under this warranty shall be at Clarity's option, without charge, of any part or unit that proves to be defective in material or workmanship during the Warranty Period.

**Exclusions from Warranty:** This warranty applies only to defects in factory materials and factory workmanship. Any condition caused by accident, abuse, misuse or improper operation, violation of instructions furnished by Clarity, destruction or alteration, improper electrical voltages or currents, or repair or maintenance attempted by anyone other than Clarity or an authorized service center, is not a defect covered by this warranty. Telephone companies manufacture different types of equipment and Clarity does not warrant that its equipment is compatible with the equipment of a particular phone company.

**Implied Warranties:** Under state law, you may be entitled to the benefit of certain implied warranties. These implied warranties will continue in force only during the warranty period. Some states do allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

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